


# AMPRIA®

## Post-Operative Rigid Dressing Instructions

---




**1.** Place the AMPRIA™ beneath the leg with the tabbed edge laid out to the side and the foam cup laid out below the leg. Align the end of your limb **at or just beyond the bottom edge of the device.**

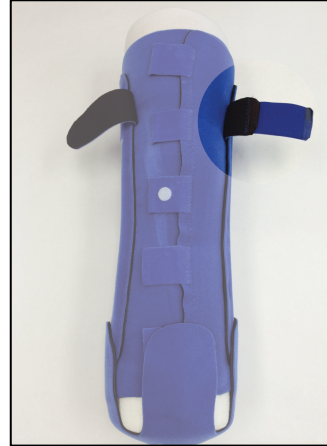

**2.** Wrap the portion with tabs around the leg and secure the marked tab first. **Beginning at top,** secure the tabs snugly along the midline of the leg except for bottom tab. The sleeve should feel comfortable as you secure the tabs.



**3.** Rotate bottom foam cup inside into contact with the limb, fasten the last tab in front over the edge of the cup and then secure the 3 large tabs gently holding the cup against the bottom of the limb

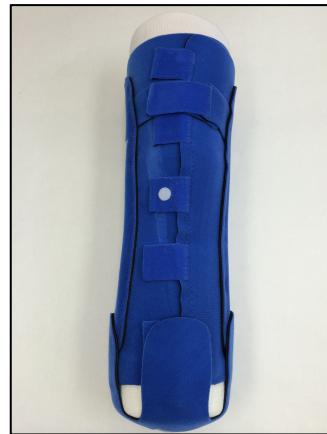

**4.** Secure the thigh strap tightly above the knee cap to aid suspension.

**5. FINISHED!** This is the appearance when correctly applied.



**QUESTIONS?** Contact us at [info@postoprotector.com](mailto:info@postoprotector.com) or (207) 831-0471.